



Bio-Therapeutic

Skin Care for life inside and out®

bt-zoom® MANUAL



Congratulations on your purchase of the bt-zoom! Your equipment has been built with the highest quality components for lasting precision and operating durability. This operation manual explains the correct use of your equipment.

Bio-Therapeutic products are not intended to treat or diagnose any medical conditions.

READ ALL INSTRUCTIONS BEFORE USE

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

BATTERY INSTALLATION

The bt-zoom requires 3 AAA batteries (included)

To change batteries, gently press in the open latch and lift the battery compartment

Remove and properly discard any old batteries

Replace batteries by following the triangular battery placement design

Gently secure the top of the battery compartment into the bt-zoom and press down on the open latch

COMPONENTS



PRECAUTIONS

Do not place or store the bt-zoom where it can fall or be pulled into a tub or sink.

Do not place in or drop into water or other liquid.

WARNINGS

The bt-zoom is not intended for use by children. Small parts may pose a choking hazard. Keep the bt-zoom away from children at all times.

Do not look directly into the bt-zoom LED lights.

Do not use the bt-zoom on the eye.

Do not view direct sunshine through the lens.

Do not place lens between direct sunlight and any combustible material.

Do not immerse in liquid.

Follow proper cleaning as outlined in the maintenance portion of this manual.

Do not open the unit for any reason; doing so will void the warranty.

Discontinue use if the bt-zoom appears damaged in any way. The bt-zoom contains no user serviceable parts. Refer to 'Guarantee and Service' if your bt-zoom no longer works or needs repair.

The bt-zoom is intended to be used as described within this manual.

LIABILITY TERMS

Bio-Therapeutic / Micro Current Technology, Inc., shall be responsible for the proper mechanical operation of the bt-zoom unit during the warranty period under the following terms:

The installation and technical repairs must be performed by personnel expressly authorized by BT/MCTI.

The bt-zoom must be used only in the manner herein described.

The bt-zoom must NOT be operated after expiration of the useful life of system-required disposable items.

The bt-zoom must only be operated using system required accessories and products described within this manual, and/or supplied by BT/MCTI.

If any of the above terms are breached, warranty on the bt-zoom unit shall immediately become null and void.

Buyer/Operator shall maintain sole responsibility for proper disinfection and timely replacement of bt-zoom system required disposable products and parts. BT/MCTI has no control over the methods by which bt-zoom is used. Therefore, BT/MCTI's sole responsibility shall be limited to repair and/or replacement of the unit ONLY under the parameters set forth. BT/MCTI, assumes no liability with regards to application or device results. Buyer/Operator herein agrees to this limited liability as a condition of sale.

TECHNICAL FEATURES

Dimensions: 8" x 3 1/4" x 1 1/2"

Weight: 3.7 oz

Power Supply: 3 AAA batteries.

MAINTENANCE

The bt-zoom should be kept free of liquid, product build up, and debris at all times. The bt-zoom case should be cleaned with a slightly damp, soft cloth and a mild cleanser. Harsh disinfectants and alcohol can damage the case of the bt-zoom and should never be used on the plastic components of the unit. The unit should never be immersed in liquid. Signs of liquid, product, corrosion, etc inside the unit will void the warranty. Should any debris appear on the bt-zoom magnification lens gently swab the lens with a clean soft cloth. The bt-zoom should be stored in a safe area at all times. The device is built to the highest quality standards however, treat it with care. Avoid dropping the device. Discontinue use if damage occurs.

bt GEAR[®]



GUARANTEE AND SERVICE

Bio-Therapeutic warrants the bt-zoom for six months after the date of purchase. Defects due to faulty materials and workmanship will be repaired or replaced at Bio-Therapeutic's expense provided that convincing proof of purchase in the qualifying period is provided.

WARRANTY EXCLUSIONS

The bt-zoom should be kept free of liquid, product build up, and What is not covered under warranty:

- Water/liquid damage.
- Damage caused by misuse, abuse, neglect, alterations, or unauthorized repair.
- Normal wear and tear, including chips, scratches, abrasions, discoloration, or fading.