



Instruction Manual

Treatment Table SL XP/PODO/HYDRAULIC



SL XP



SL XP PODO

Hydraulic models without illustration.

Gharieni Group GmbH
Am Schürmannshütt 24
D - 47441 Moers


➤ Phone:	+49 (0)2841/88300-0
➤ Fax:	+49 (0)2841/88300-999
➤ Customer service hotline:	+49 (0)2841/88300-40
➤ E-mail:	service@gharieni.de
➤ Web:	www.gharieni.de

Index

1. Safety information.....	3
2. General information	5
2.1 Basic models.....	5
3. Prefaces.....	6
4. Intended use.....	6
5. Delivery	6
6. Unpacking.....	7
7. Before initial operation.....	7
8. Rating plate	7
9. Installation.....	7
9.1 Headrest.....	8
9.2 Legrest.....	8
10. Adjustment of the SL	9
10.1 Manual control panel	9
10.2 Vibro - Manual control panel.....	10
10.3 Backrest.....	10
10.4 Legrest.....	10
10.6 Inclination.....	11
10.7 Armrests	11
10.8 Work height.....	11
10.9 Memory-functions.....	12
11. Initialisation of the software	12
12. Options.....	13
12.1 Foot control panel.....	13
12.2 Upholstery	14
12.3. Airback.....	14
12.4 Heater Smart-Thermo.....	15
12.5 Audio-Relax.....	17

12.6 Rotunda	18
12.7 Vibro Wellness 10	19
13. Repair, maintenance and cleaning.....	22
14. Guarantee provisions.....	23
15. Waste management	23
16. Technical data.....	24
17. EC-Declaration Of Conformity	25
18. Status of edition.....	26

Abbreviations

IM	Instruction Manual
MCP	Manual control panel
FCP	Foot control panel
HCP	Heater control panel
	Operating advice



Do not dispose of this appliance together with your domestic waste. The appliance must be disposed of at a collecting centre for recyclable electric and electronic appliances. Do not remove the symbol from the appliance.

1. Safety information

READ THE COMPLETE SAFETY INSTRUCTIONS CAREFULLY BEFORE YOU OPERATE THE APPLIANCE FOR THE FIRST TIME. All information included in those pages serve for the protection of the operator. If you ignore the safety instructions, you will endanger your health and life.

⚠ DANGER! indicates a hazardous situation which, if ignored, will result in death or serious injury.

⚠ CAUTION! indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.

⚠ WARNING! indicates a hazardous situation which, if ignored, could result in death or serious injury.

NOTICE! indicates possible damage to the appliance.

Store this manual in a safe place so you can use it whenever it is needed. Strictly observe the instructions to avoid damage to persons and property.

DANGER


1. All electrical work to connect the SL to the mains must be carried out by a qualified professional. Do not modify the energy supply. The connection must be carried out in accordance with the current local and legal regulations.
2. **Do not connect** the SL to the mains if the SL itself or the power cable or the plug are visibly damaged.
3. Never try to repair the SL yourself. If the SL does not operate properly, please contact the manufacturer or the shop you purchased the appliance at. Original spare parts should be used only.
4. When the power cable is damaged, it must be replaced by the manufacturer or an authorized aftersales service or a qualified professional only.

WARNING!

1. Use a properly-grounded safety socket with proper voltage only. The electrical components must never come into contact with water. Check all cables regularly and call the aftersales service when any cable is in improper condition. **RISK OF ELECTRIC SHOCK!**
2. Do not use water or any other liquid for treatments. Do not pour water or any other liquid into the SL! **RISK OF ELECTRIC SHOCK!**
3. For long periods of use, always keep some free space between the back and seat upholstery. **RISK OF BURNS DUE TO HEAT ACCUMULATION!**



Right

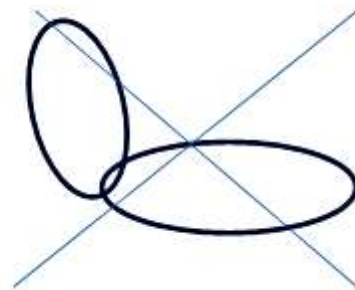

Wrong

4. Persons suffering from acute pain, venous disorders, low blood pressure, inflammatory processes and other pathologies which make heat-treatments or the positioning of the patient inappropriate must consult the doctor before treatment.

5. Persons who are treated on the SL must not be under the influence of intoxicating substances (e.g. alcohol, drugs or medicine)
6. Always keep children away from the SL. Children must not play with the PS or its single components.
7. This device is not intended to be operated by persons with reduced physical, sensory and mental capabilities or lack of experience or knowledge, unless they are supervised or instructed concerning the use of the device and do understand the danger resulting thereof.

2. General information

Dear Customer,

we would like to thank you for purchasing this quality-product as well as for your confidence. **Read the complete IM before you use the SL for the first time.** Reading the complete IM is essential for its proper and confident use and will help you to operate the appliance to your entire satisfaction. Our SL is very robust and has a long operating life if it is operated properly.

Read the instructions very carefully and operate the appliance in its intended use as described only to protect you and your customers from danger.. Keep the IM handy and near of the SL.

⚠️WARNING! Failure to follow these instructions endangers the operator's health and life and/or can damage the appliance.



Ensure that there is sufficient free space for the SL relevant to its total length and rotation.

2.1 Basic models

SL XP 1	Electric height adjustment. Backrest, legrest, inclination via gas pressure spring.
SL XP 2	Electric height adjustment + either backrest or inclination. Remaining functions via gas pressure spring.
SL XP 3	Electric height adjustment + backrest + inclination. 3 memory positions. Legrest via gas pressure spring.
SL XP 4	Fully-electric adjustment. 3 memory positions.
SL XP PODO 1	Electric height adjustment. Backrest, legrest, inclination via gas pressure spring.

SL XP PODO 2	Electric height adjustment + either backrest or inclination. Remaining functions via gas pressure spring.
SL XP PODO 3	Electric height adjustment + backrest + inclination. Legrest via gas pressure spring.
SL XP/PODO Hydraulic	Hydraulic height adjustment. Pneumatic adjustment for backrest, legrests and inclination. Rotatable seat (360°).

3. Prefaces

1. The SL is subject to the highest safety and quality standards.
2. Before initial operation, please check that all components are present and that they are not visible damaged.
3. The dedicated safety socket must be grounded properly and must meet the proper voltage.

4. Intended use

Read the instructions very carefully and only operate the device in its intended use as described to protect you and your customers from danger. The SL is intended for indoor use only.

The SL is suitable for facial treatments, hair removal, pedicure and body treatments.

5. Delivery

⚠️WARNING! While unpacking, the packaging materials (polythene bags, polystyrene pieces, etc.) should be kept away from children. CHOKING HAZARD!

Before installation, ensure that there is sufficient free space for the SL relevant to its total length and rotation. In some countries, the SL is delivered in a cardboard.

The legrest is folded by 90° (see picture below).



6. Unpacking

Please observe the following when the SL is delivered in a cardboard:

- Open the top of the packaging carefully. Do not use any sharp-edged items which can damage the upholstery.
- Do not lift the SL but remove the top packaging first. Then you can lift and remove the outer packaging
- The headrest is in the same cardboard. The headrest has to be installed (see chapter 9).
- Carry the SL using its feet only.. Remove the transport protection (2 safety straps) before initial operation.


NOTICE! If the floor is not level, you must ensure proper stability using the adjusting screw.

7. Before initial operation

Before initial operation, please check that all components are present and that they are not visibly damaged.

Before installation, ensure sufficient free space: Min. free space required is: Length = 204 cm * Width = 80 cm.

8. Rating plate

 **WARNING!** The rating plate is on the casing of the foot. Do not make the rating plate illegible! Do not remove the rating plate! Failure to do so will expire the guarantee.

9. Installation

The SL is easy to install and can be adapted to all human body measurements. Depending on model, you can adjust the desired positions electrically or pneumatically in a very short period of time and then focus entirely on your customer.

9.1 Headrest

You find the headrest in the cardboard of the SL. Insert the chrome pins of the headrest into the supports on the top of the backrest until they lock. You can adjust the height of the headrest using the dedicated locking levels relevant to the height of the customer. You can remove the headrest completely by extracting it upwards.



To provide an opening for the nose (in massage position), extract the headrest by approx. 10 cm.

A piece of adhesive strap is on the back of the upholstery of the headrest to attach the optional available neck and headrest "Dentalax" which will increase the comfort of lying significantly.

9.2 Legrest

You can extract the lower part of the legrest to increase its length.



Furthermore, you can remove the lower part, so you can work on the heel area (pedicure). This setting is also suitable when you want to carry out massages without using additional tools (lowered foot position).

PODO-models: These models have two separate legrests.

10. Adjustment of the SL


The adjustment modes are depending on the model you have opted for. Please refer to chapter 2.1.

10.1 Manual control panel

The MCP¹ is a wired control operating with low voltage (24 V). The electrically controllable functions are controlled via the MCP. Depending on model, the functions are controlled via the corresponding pair of push buttons. How to save specific lying positions is described in section **memory functions**.

1 SL Hydraulic-models are not equipped with an MCP.

⚠ WARNING! Ensure that no parts of the body or items are between the parts of the SL while adjusting the lying position. **RISK OF CRUSHING!**
NOTICE! For safety reasons, the SL moves only while pressing a key of the MCP. When you stop pressing, all moving-operation will stop immediately.

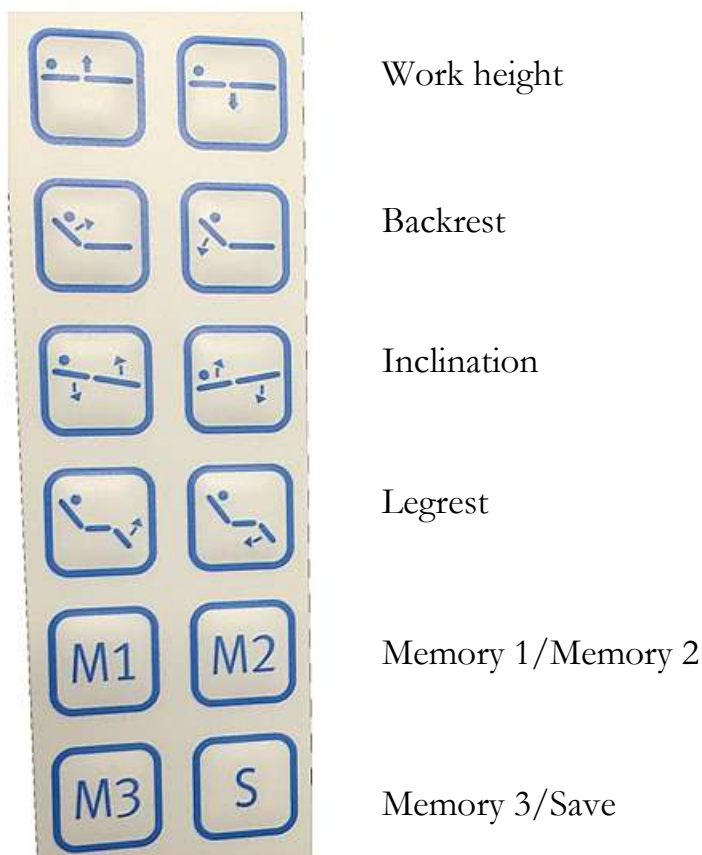


10.2 Vibro - Manual control panel


If you have opted for the VIBRO-equipment, an additional manual control panel is included in delivery. Look at chapter 12.7 for further information.

10.3 Backrest

Electric adjustment: The inclination can be adjusted via the MCP. The motors for the height and backrest adjustment are equipped with the SPLINE-function. This function causes the motors to run the lifting movement only, the lowering movement is performed by dead weight.

Pneumatic adjustment: The adjustment is performed pneumatically via a gas pressure spring. The lever is installed on the right side of the seat.

 **WARNING!** The SPLINE-function does not protect from injuries. So it is essential that no parts of the body or items are between the parts of the SL while adjusting the lying position. RISK OF CRUSHING!

 **WARNING!** While adjusting, always check that fingers cannot be caught between the backrest and the seat. These two locations are marked by a yellow warning triangle.

10.4 Legrest

Electric adjustment: The legrest can be adjusted individually. You have reached the highest position, when the legrest is level with the seat. The adjustment is performed electrically (via MCP). An additional foot control panel is optional available.

Pneumatic adjustment: The adjustment is performed pneumatically via a gas pressure spring. The lever is installed on the right and left side of the relevant legrest.

10.6 Inclination

The inclination of the seat is fully adjustable.



Electric adjustment: Press the relevant push button of the MCP:

Pneumatic adjustment: The adjustment is performed pneumatically via a gas pressure spring.

10.7 Armrests

The armrests are always in parallel to the seat. You can fold back the armrests for easy access to the seat.

10.8 Work height

- The work height is fully adjustable from 59 cm to 90 cm (Hydraulic: 63-83 cm).
- Installing of the Rotunda-function or rollers will increase the work height a bit.
- The entry height is the lowest position of the seat (59 cm/seat in horizontal position).
- Usually, the SL is entered from the side. Fold back the armrests for easy access. As the legrest can be adjusted by 90°, it is also possible to enter the SLP from the front. This mode is advantageous, when access from the side is not ensured.

10.9 Memory-functions

The availability of the memory functions is depending on the model you have opted for. Please refer to chapter 2.1.

The memory function saves your desired treatment positions. Depending on model, press the push buttons M1, M2 or M3.

How to save a position:

1. When the desired position has been reached, press button "S" for approx. one second to save the position.
2. You can select the memory button now:
3. Press M1, M2 etc. An acoustic signal indicates successful saving.

Activation of a memory position:

Press the relevant push button M1, M2 etc. Keep pressing until the position has been reached.

11. Initialisation of the software

If a previously-saved position is no longer properly set, you may have to reinstall the software. This may occur after some 100 times of use. To correct the differences of position, you have to carry out the following steps:

- The backrest motor must be set to max position.
- The other motors must be set to min. position.
- The adjustment (initialisation) must be carried out manually, i.e. each motor has to be run individually.
- When the final positions are set, please keep pressing the push buttons for additional 2 to 3 seconds.

12. Options


The availability of the options is depending on the model.

12.1 Foot control panel


You can connect an additional FCP. One motor can be adjusted using the buttons of the FCP. The motor you can adjust is depending on model.

For all models: When you operate two buttons simultaneously, you activate previously-set memory positions.

FCP “Model Podo”

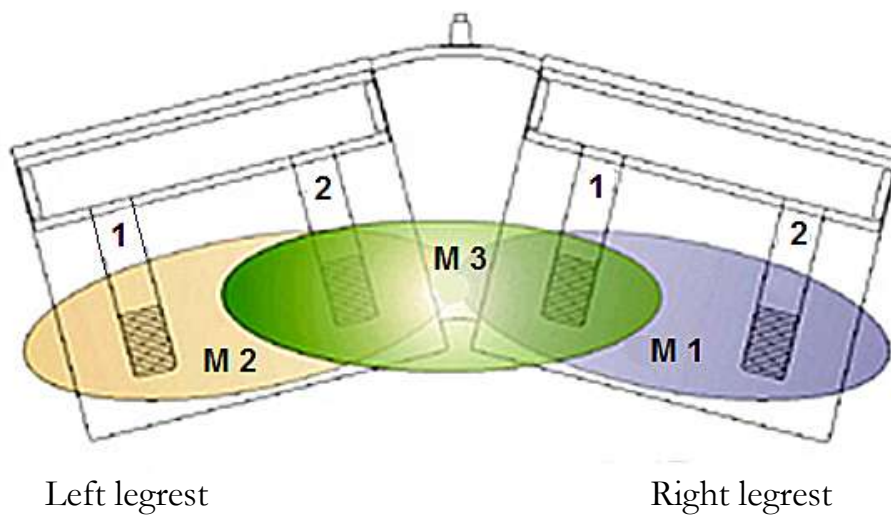

FCP “Model Cosmetic”

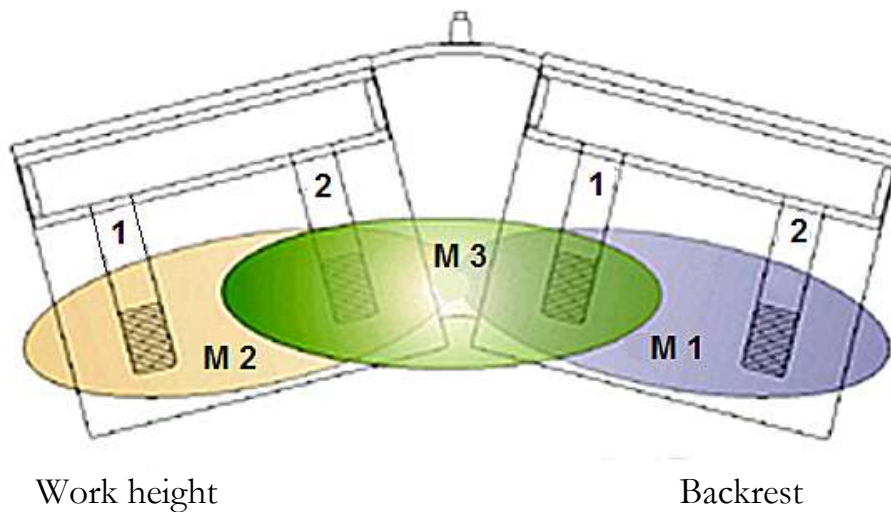

1	Up
2	Down
M1	Memory 1
M2	Memory 2
M3	Memory 3

12.2 Upholstery

The **standard upholstery** is anatomically shaped and equipped with an **AIRBACK®** for lumbar support. The surface is made of artificial leather, available in many colours of our two colour collections. You can also combine several colours and - as an option - select the colour of the thread. Moreover, the armrests of the standard upholstery are equipped with mounting holes for the installation of manicure bowls.

12.3. Airback

Airback, manual

This option is standard for all SL-upholsteries.

The backrest (lumbar section) is equipped with an air cushion. You can inflate the air cushion using the grey hand pump. The airback is useful when your customer is placed on the SL in supine position. The airback supports the human anatomy (S-shaped spine) and increases the lying comfort, especially when treating persons with a hollow back (in this case, please pump some more air into the airback).

When the customer is placed in prone position, do not use the airback. You can deflate the airback using the black button of the grey hand pump.

Airback, electric

As an option, the airback can be operated electrically. The control buttons are installed on the casing of the seat. Use the left button to inflate the air cushion. Use the right button to deflate the air cushion. The air cells are electrically-protected from overfilling.

12.4 Heater Smart-Thermo

WARNING

1. Do not use the option Smart-Thermo when your customer
 - has got a pacemaker.
 - is pregnant.
 - is affected by thrombosis.
 - suffers from pain due to unknown reasons.
 - Apply to healthy people only.
2. Do not cover the SL with blankets, towels, clothes etc. while the heater is running; otherwise heat accumulation can occur. **RISK OF FIRE!**
3. Do not exceed the default values set by the manufacturer (pre-heating period approx. 6 minutes; treatment heating-level approx 30%, 3 LEDs) when programming the heater yourself.
4. When the treatment is stopped prematurely, cancel the Smart-Thermo function by pressing the OFF-button on the HCP to avoid heat accumulation.
5. Disconnect the SL from the mains when it is not used for a longer period of time.

Heater

The lying surface can be heated to maximize the treatment comfort. You can set 10 heating levels using the HCP. For quick heating of a cold lying surface, after starting the heater will operate with max. power during the pre-heating period. The parameters of the heater can be programmed and saved if desired. Check the current temperature before using the SL. Disconnect the SL from the mains when it is not used for a longer period of time, e.g. during the night.



HCP

➔ Operation

You can operate the heater using the 4 buttons ON/OFF/+/- .The setting of the heating power is indicated by a 10-level LED bar-diagram.

The green LEDs 1-4 indicate low power/low temperatures.

The yellow LEDs 5-7 indicate medium power/medium temperatures.

The red LEDs 8-10 indicate high power/high temperatures.

➔ Heating

Switch on the heater using the ON-button on the HCP. The preheating period (6 minutes-default setting) starts on highest setting. All LEDs are on. When the pre-heating time has elapsed, the normal treatment setting will start (60 minutes-default setting). You can stop the pre-heating period using the + and - buttons and set the temperature to one of the 10 levels. Switch off the heater using the OFF-button on the HCP.

➔ Default settings

Total heating time	60 minutes	LED 6
Pre-heating period	6 minutes	LED 1+2
Power/total operation	30%	LED 1+2+3

➔ Programming

The values of uninterrupted operation, the pre-heating period and the heating power are programmable.

Use the following combination to enter the programming mode:

1. Press and hold the OFF-button.
2. Press and release the ON-button.
3. Release the OFF-button and enter the

➔ Setting of the total operation period

You can set 10 levels from 10 to 100 minutes. The position of the single and illuminated LED - from the left to the right - indicates the level. The manufacturer's setting is 60 minutes.

Example for level 6 = 60 minutes total operation



Press the ON-button to save your setting.

➔ **Setting of the pre-heating period**

Each illuminated LED indicates a pre-heating period of 3 minutes. The manufacturer's setting is 6 minutes. The max. pre-heating period available is 30 minutes. Do not exceed the default values set by the manufacturer.

While the heater is still warm, the pre-heating period may be shorter when the heater is switched on again.

Press the ON-button to save your setting, then you enter the

➔ **Setting of the power during treatment**

The currently-saved value is indicated by one LED bar. The number of illuminated LEDs indicates the set heating power in 10% steps. Manufacturer's setting is 30%. Set the desired value by pressing the + and - buttons. Save your setting by pressing the ON-button.

➔ **The programming of the heater is complete**



You can finish the programming of the heater by pressing the OFF-button at any point of time.

12.5 Audio-Relax

This option supports optimal relaxing of your customers and adds an individual touch to your massages. Two speakers laterally-installed in the headrest support the massage by a unique experience of sound without harassing your customer. The

MP3-player is equipped with 50 minutes of GEMA-free relaxing-music. Moreover you can load other music on the MP3-player if desired. The volume is not intended for filling rooms with sound.

12.6 Rotunda

This option enables the seat to be rotated by up to 200° (Podo models) or up to 330° (Cosmetic models). You can freeze the set position for treatment using the hand lever which is located below the seat. Loosen the hand lever by one rotation of the hand in clockwise direction, rotate the seat to the desired position and tighten the lever by turning it in anti-clockwise direction.



⚠ WARNING! Hold the top of the seat only while rotating. Do not grasp below the metal rotating-disk. RISK OF CRUSHING!

NOTICE! You cannot use the Rotunda-function when the legrest is folded by 90° . If the legrest contacts the base while rotating, please fold up the legrest slightly.

12.7 Vibro Wellness 10

⚠ WARNING

1. Do not use the option Vibro Wellness 10 when your customer
 - has got a pacemaker.
 - is pregnant.
 - is affected by thrombosis.
 - suffers from pain due to unknown reasons.
 - Apply to healthy people only.
2. Do not open the housing of the manual control panel! Failure to do so will expire the guarantee.
3. Liquids must not get into the manual control panel.
4. **Persons with a pacemaker must not operate or use the massage unit as the pacemaker can be negatively influenced.**

The Vibro Wellness 10 has 10 motors in different sections (fig. 1).

Use the Vibro manual control panel (fig. 2) to activate the individual sections in pairs.

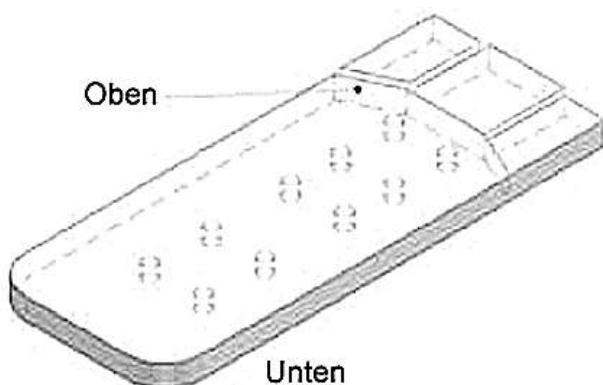


Fig. 1

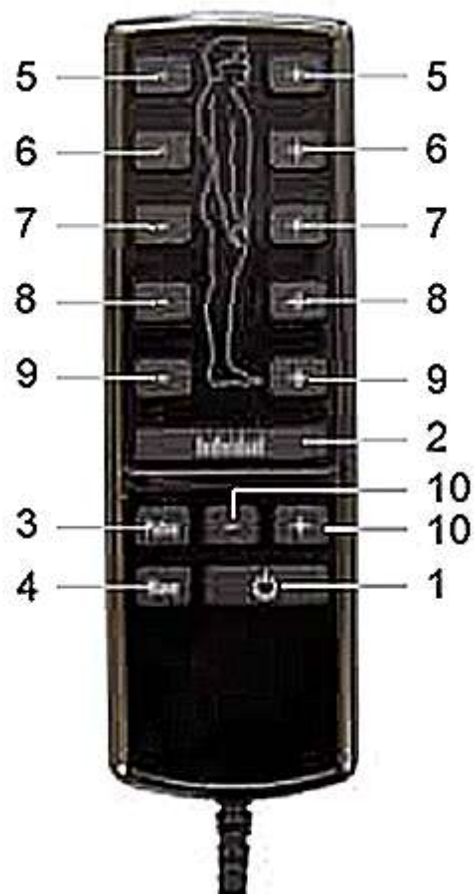


fig. 2



Operation

1 Starting of a massage.

You can start a massage by pressing the following buttons:

1.1 ON/OFF-button (1).

When the massage unit is operated for the first time, the setting "individual" is activated. All massage motors operate in the manufacturer's setting (intensity level 2; medium intensity). When the massage unit has been operated before, the previously-set program is started again (individual, pulse or wellness massage).

1.2 Individual button (2).

When the massage unit is operated for the first time, the manufacturer's setting is activated (intensity level 2; medium intensity). When an individual setting has been set before, the previously-set setting is started again.

1.3 Pulse-massage button (3).

Start this massage with the first of the three pulse-massage programs.

1.4 Wellness-massage button (4).

Start this massage with the first of the three wellness-massage programs.

1.5 Intensity buttons (5-9).







Pressing one of these buttons sets the relevant massage motor to intensity level 1. Please note: doing so will clobber a previously-set individual setting (look at § 3, individual settings)

2 Finishing of a massage.

A timer finishes a massage after 15 minutes automatically. If you want to stop the massage before, press the ON/OFF-button (1).

3 Individual setting.

Start the massage by pressing the individual button (2). When the massage unit is operated for the first time, the manufacturer's setting is activated (intensity level 2; medium intensity). Now you can set each motor to three levels of intensity with the buttons 5-9. The levels are indicated on the LED-display as follows.

	Level 0 (off)
	Level 1
	Level 2
	Level 3
	LED off
	LED on

When the lowest intensity (level 1) has been reached, press one of the buttons 5-9 again to switch off the relevant motor. When you switch off all massage motors, the massage is stopped. You can change from the manufacturer's setting (intensity level 2; medium intensity) to your individual setting by pressing the individual-button (2). The individual settings are saved after the massage.

4 Pulse-massage.

Press the pulse-button (3) to start the massage with the first of the three programs. Press again to start the second program. Press a third time to start the third program. When you press the button (3) a fourth time, the first program is started again. You can select three levels of speed with the + and - buttons (10). You can select three levels of intensity with the buttons 5-9. Please note: The pulse massage will be restarted in the manufacturer's setting.


5 Wellness-massage.


Press the Wave-button (4) to start the massage with the first of the three programs. Press again to start the second program. Press a third time to start the third program. When you press the button (4) a fourth time, the first program is started again. You can select three levels of speed with the + and - buttons (10). You can select three levels of intensity with the buttons 5-9. Please note: The wellness-massage will be restarted in the manufacturer's setting.



The motor of the Vibro option becomes loud when operating the highest setting. This does not constitute a reason for complaint and is not a reason for concern.

13. Repair, maintenance and cleaning

 **WARNING** Disconnect the SL from the mains before cleaning and maintenance.

 **WARNING** Water must never penetrate the housing.

We are responsible for safety and reliability of the SL under the following conditions only:

1. Repairs and modifications must be carried out by Gharieni Group GmbH or a person authorized by Gharieni Group GmbH.
2. The electrical installation of the room the SL is installed in must meet the current legal regulations at the site where the SL is operated.
3. The SL is operated as described in the instruction manual.
4. Unauthorized modifications or manipulating the SL will void the liability of the manufacturer.
5. All electric components are designed for continuous operation. Leave the SL connected when it is used every day.
6. Check the power cord regularly for signs of damage.
7. When the power cable is damaged, it must be replaced by the manufacturer or an authorized aftersales service or a qualified professional only.

Depending on model, the upholstery is made of PU (polyurethane) or synthetic leather.

- ❖ Clean the **PU** using a cloth provided with a small amount of alcohol.
- ❖ Clean the **synthetic leather** using the special detergent for synthetic leather (optional available from Gharieni).
- ❖ Clean the housing and the fittings using a damp cloth and a small amount of soap.

14. Guarantee provisions

The Gharieni Group GmbH supplies all new equipment subject to a 2-years guarantee. Wearing parts (such as e.g. incandescent lamps, UV lamps, glass panels, consumables etc.) are not covered by this guarantee. Damage or loss due to unprofessional or improper use, non-compliance with the provisions of the instructions for use and/or gross negligence is not covered by guarantee either. In addition, the guarantee is invalidated if any stickers (such as e.g. the type plate, instruction notices etc.) are removed or if appliances or parts of appliances are opened without this being explicitly allowed in the operating instructions (e.g. for cleaning).

Guarantee exclusion:

Upon delivery, it is necessary to check that the equipment has not been damaged during transportation and that the set of accessories is complete and in good condition. Claims must be filed within 8 days after the delivery of the Gharieni SL. Apart from the cases specified in the supply agreement, **the guarantee is void when:**

- The malfunctioning is attributable to the operator.
- The instructions in this instruction manual are not followed.
- The Gharieni PLS has suffered changes after repair operations performed by a third party unauthorized by Gharieni Group GmbH.

15. Waste management



- Do not dispose of the SL together with your domestic waste. Old and unused appliances must be sent for disposal to the responsible recycling centre for electric appliances. Do not remove the relevant symbol from the product.
- The SL is made of metal and plastic components, electronic components, electric cables. Dispose of the separate components in accordance with the current local disposal regulations. For further information, please contact your municipal disposal authorities, your retailer or

Gharieni Group GmbH

- | | |
|-----------------------------|-----------------------|
| ➤ Phone: | +49 (0)2841/88300-0 |
| ➤ Fax: | +49 (0)2841/88300-999 |
| ➤ Customer service hotline: | +49 (0)2841/88300-40 |
| ➤ E-mail: | service@gharieni.de |

16. Technical data

Total length	144-204 cm
Max. width	80 cm
Width without armrests	62 cm
Range of height adjustments incl. adjustable feet	59- 90 cm Hydraulic: 63-83 cm
Range of height adjustments incl. rollers	61-102 cm Hydraulic: 65-85 cm
Max. height incl. backrest 75°	174 cm
Inclination of backrest	0° to 75°
Lying surface lifting capacity ¹ (max. safe operational load)	Models 1/2: 135 kg Models 3/4: 160 kg
Backrest lifting capacity ¹ (max. safe operational load)	75 kg
Total weight	approx. 83 kg
Power of transformer (primary)	230V AC/50Hz/200 VA max.
Power of transformer (secondary)	24 V DC
Protection	3 ampere AT
Duty cycle (drive)	10%, max. 2 minutes of permanent operation/18 minutes rest
Power cord	2.6 m, straight, black
MCP cable	Coiled cable, grey
Device protection class	II
Overvoltage protection class	Type B
Protection against solid and liquid materials	IP 51

1 Incl. safety factor

Ambient conditions for storing and transportation

Temperature	from -10 °C to +50 °C ambient temperature
Relative humidity	from 20% to 90% at 30 °C non-condensing

Ambient conditions for operation

Temperature	from +5 °C to +40 °C ambient temperature
Relative humidity	from 35% to 65% at 30 °C non-condensing

17. EC-Declaration Of Conformity

EC DECLARATION OF CONFORMITY

We

Gharieni Group GmbH

(Manufacturer)

**D-47443 Moers
Gutenbergstr. 40
Germany**

declare that the product

(Type or model, name)

CE

SL-Series

2017

is produced in accordance with the requirement specified in

Low Voltage Directive (LVD) 2014/35/EU

applied standards: EN 60335-1

Electromagnetic compatibility 2014/30/EU

applied standards: EN 55011:2017-03, EN 55022:2011-12

EN 61000-3-2:2015, EN 61000-3-3:2014

EN 61000-4-2:2009, EN 61000-4-3:2011, EN 61000-4-4:2013, EN 61000-4-5:2015, EN 61000-4-6:2014

EN 61000-4-8:2010, EN 61000-4-11:2005

Safety of Machinery 2006/42/EG

applied standards: EN 12100:2013

RoHS Directive 2011/65/EU

applied standards: EN 50581:2013

Moers, 03/15/2017

Place and date



S. Gharieni
Executive president

18. Status of edition

Edition	Alterations (pages)	Date
First edition	-	04/22/2016
Second edition	24	04/28/2016
Third edition	25	04/04/2017
Fourth edition	1 / 26	06/14/2017



NOTES

THE PROPERTY RIGHTS OF THIS INSTRUCTION MANUAL ARE RESERVED. ITS REPRODUCTION OR TRANSMISSION TO THIRD PARTIES IS FORBIDDEN WITHOUT THE AUTHORISATION OF THE OWNING COMPANY. THE POWER TO PROTECT ITS RIGHTS IS RESERVED AND OFFENDERS SHALL BE PROSECUTED.

Gharieni Group GmbH
Am Schürmannshütt 24
D - 47441 Moers