

DÉESSE<sup>PRO</sup>



DÉESSE PRO

— OPERATING MANUAL —

CE

# DÉESSE PRO

GLOBAL DISTRIBUTOR

*hgi*

Harpar Grace

INTERNATIONAL

Harpar Grace International  
Highland House, Mayflower Close,  
Chandlers Ford, Hampshire, SO53 4AR

Tel: 02038 686242

[enquiries@harpargrace.com](mailto:enquiries@harpargrace.com)

## CONTENTS

The Components	3
Components Explained	4
Operating Instructions	5
Treatment Programmes	6
Specifications	7
Warranty	8
Cautions	8

# DÉESSE PRO

## THE COMPONENTS




Déesse PRO LED Mask




Protective Eyewear



Control Panel

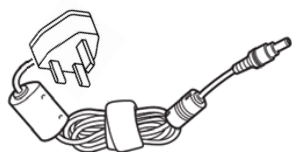


Power Cable

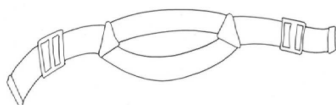

Retaining Strap

*Not required if laying  
down during treatment*

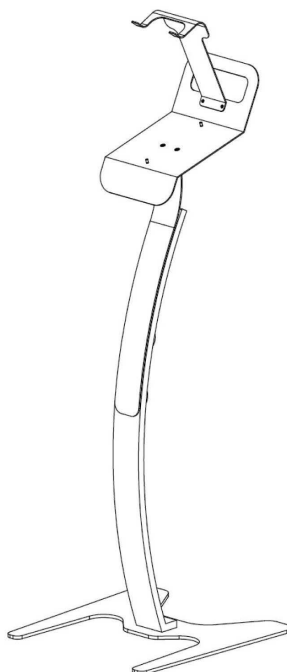

Déesse PRO Mask Stand  
*(Optional extra)*

# COMPONENTS


## EXPLAINED

### Mask Components

**Front**




**Reverse**

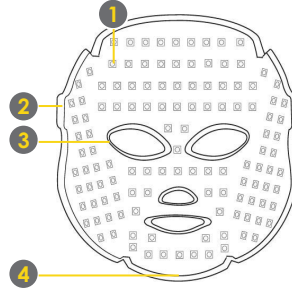


1	Surface mounted LED's	To emit the required wavelength of light
2	Strap connecting holes	To secure the retaining strap to the mask if required
3	Soft rubber eye protector	For safety and comfort during treatment
4	Power Cable	Power Cable to connect the mask to the control panel via the USB port

### Control Panel Components

**Front**

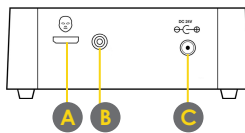



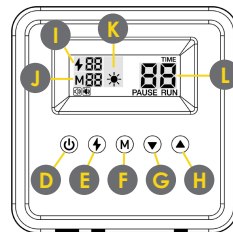



**Top**



A	Mask connecting USB port	Port to connect the control panel to the mask via the USB cable
B	Power indicator	Bulb illuminates blue when power is connected
C	Power cable port	Port to connect the control panel to the electricity supply
D	Power button	To turn the mask ON and OFF or to PAUSE treatment
E	NIR selector	To engage or disengage Near-Infrared (NIR)
F	Mode selector	To select the required treatment mode
G	Decrease minutes button	Reduce the required treatment duration
H	Increase minutes button	Increase the required treatment duration
I	NIR indicator	Indicates if NIR is selected by <b>Button E</b>
J	Mode indicator	Indicates the mode selected by <b>Button F</b>
K	Action display	Ticks around indicating the programme cycle is running
L	Running time	Indicates time selected or time remaining for the programme

# DÉESSE PRO

## — OPERATING INSTRUCTIONS —

1. Connect the appropriate end of the 'power cable' into power cable **Port C** at the front of the control panel, and plug the opposite end into the electric wall socket. **Light B** will illuminate indicating that power is being supplied to the control panel.
2. Connect the appropriate end of the 'mask to control panel cable' into **Port A** at the front of the control panel.
3. Press and hold **Button D**  for 2 seconds, and release when you hear a short melody. The LCD screen on the top of the control panel will illuminate. The control panel is now ready for programming.
4. **MODE SELECTION** (see pages 6-7) : Repeatedly press and release **Button F**  until the desired mode is selected.
5. **NEAR-INFARED (NIR)** : Press **Button E**  to set NIR off or on for your treatment.
6. The treatment duration is pre-set, but this can be changed using **Button G**  to decrease treatment time (*minimum 5 mins*), or **Button H**  to increase it (*maximum 30 mins*).
7. Eye protection should be worn over the eyes and under the mask for safety. Place the mask gently over the face. Make sure it is comfortable. The mask can be repositioned during treatment in the event of any discomfort.
8. If laying down, the retaining strap does not need to be worn, but if you wish to sit, then the strap will hold the mask in place. **NB. Please take care not to fasten the straps too tightly!**
9. Press and release **Button D**  to start treatment. The same button can be pressed and released during treatment to PAUSE it, then pressed once again to continue.
10. When the set time has elapsed, the programme will automatically end with a short melody and the LED lights will turn off.
11. The mask should be gently but thoroughly cleansed and disinfected after every treatment using anti-bacterial cleansing wipes (or similar), then allowed to air dry.

**NB. Please handle your device and its' components with the greatest of care**

# DÉESSE PRO

## PROGRAMMES

### Anti-Ageing



#### Mode 1

Red

*\*\*See comment for use of 630nm\*\**

### Purifying



#### Mode 2

Blue

### Brightening



#### Mode 3

Green

### Post-Procedure



#### Mode 4

Near Infrared & Blue & Red

## PROGRAMMES

### Calming



**Mode 5**  
Blue & Green

### Anti-Ageing Express



**Mode 6**  
Red

*\*\*See comment for use of 630nm\*\**

**\*\* NB: Modes 1 & 4 utilise 154 red LEDs (630nm). Mode 6 utilises 308 red LEDs (630nm) as express setting.**

# DÉESSE PRO

## SPECIFICATIONS

Product	Déesse PRO (LED Face Mask)
Function	To treat numerous skin concerns
LED Wavelengths	Blue (415nm), Green (520nm), Red (630nm) and Near Infrared (830nm)
Quantity of LED's	154 of Blue (415nm), Green (520nm), and NIR (830nm). 308 of Red (630nm) Total 770 bulbs
Input	100 - 240VAC, 50/60 Hz, Max 20W
Output	24VDC, 1A

# DÉESSE PRO

## WARRANTY

- Product must be registered online within one month of purchase for warranty to be valid
- **Register at [www.deessepro.com/warranty](http://www.deessepro.com/warranty)**
- Warranty is valid for 12 months following date of purchase.
- Should your product become faulty within the warranty period please contact [enquiries@harpargrace.com](mailto:enquiries@harpargrace.com) or the company you originally purchased the product from.
- To view our warranty policy visit [www.deessepro.com/warranty-policy](http://www.deessepro.com/warranty-policy)

# DÉESSE PRO

## CAUTIONS



### OPERATING CAUTIONS

- Use only with the power cables supplied.
- Do not place any wet objects near the device.
- Do not operate with wet hands.
- Do not try to open or tamper with any part of the device.
- Do not make direct eye contact with LED light when device is in operation. Excessive exposure to the light can cause eye damage.
- To protect your eyes from Infrared exposure, wear Protective Eyewear before LED irradiation and remove once the appliance is turned off.



### TREATMENT CAUTIONS

- Do not use the mask until you have read the instructions.
- Do not use whilst pregnant
- Do not use whilst being treated for cancer
- Seek medical advice in case of epilepsy or hypersensitivity to light.
- Do not look directly at the bulbs when in use.
- May be used daily if required.



### STORAGE

- Store away from high temperatures and moisture.
- Clean and disinfect with antibacterial wipes after each use.
- Handle with care to avoid accidental damage.

After reading this manual, be sure to keep it in a safe place where the user can always access it.

This appliance is intended to be used in households, salons, spas and clinics.